



User Manual

Identification rods for embryo transfer straws

Lire attentivement ce manuel

Edition: 01/13/2022

avant d'utiliser

007334 / 006470 / 006424 / 006328 / 006326 / 006329 / 006327 / 007347 / 007346

IMV TECHNOLOGIES – www.imv-technologies.com – tel +33 (0)2 33 34 64 64

Ce document ne peut être reproduit, copié, transmis ou donné à une partie autre que l'utilisateur final sans l'autorisation écrite explicite d'IMV Technologies.

ID rods can be supplied in two lengths:

- 1) 13 mm → To fit 133 mm ET straws:
 - Ref 007342: 13 mm rods yellow color **sterile** by 40 units (8 blisters of 5 ID rods)
 - Ref 007352: 13 mm rods yellow color non-sterile by 200 units

- 2) 48mm → To fit 91 mm and 133 mm ET straws
Please note when using 48 mm rods on 133 mm straws the whole length (181mm) would be higher than a classical storage goblet (115mm); care should be taken to avoid storage goblets on top of each other.
 - Ref 007334: 48mm ET rods yellow color **sterile** by 40units (8 blisters of 5 ID rods)
 - Ref 006470: 48mm ET rods yellow color non-sterile by 100 units
 - Ref 006424: 48mm ET rods white color non-sterile by 100 units
 - Ref 006328: 48mm ET rods blue color non-sterile by 100 units
 - Ref 006326: 48mm ET rods pink color non-sterile by 100 units
 - Ref 006329: 48mm ET rods green color non-sterile by 100 units
 - Ref 006327: 48mm ET rods orange color non-sterile by 100 units
 - Ref 007347: 48mm ET rods salmon color non-sterile by 100 units
 - Ref 007346: 48mm ET rods grey color non-sterile by 100 units

Instructions for labeling embryo straw:

- 1) Open package under laminar flow hood
- 2) Follow labeling guideline according to IETS recommendations:
<https://www.aeta.org/docs/AETAstrawlabel.pdf>
From IETS Manual, 4th Ed., Ch. 9, pg. 94
- 3) It should include the following fields:

Printed item	Example printed on straw
Straw number	2
Process code	DT
*Stage n° - Quality	5-1
Practitioner code	E 515
Donor breed	HO
N° of embryo in straw	1
Donor Reg N°	123456789
Freeze date	YYMMDD
Donor tattoo	IRIS
Sire breed	HO
Sire reg	1234567
*Sire name	IMV
*Optional	

- 4) We recommend the use of a printer as IRIS compact ref 028865 to print the ET straw and the use of permanent ink pen to write on the ID rod (STAEDTLER permanent pen Nr 310CDS-9) ref: 014175

- 5) We do not recommend using sticking labels on straws and rod. sticking material on label could varies between batches and printed label may be lost during storage in liquid nitrogen .
- 6) We strongly recommend to run a test when you switch to a different permanent marker pen to make sure that the ink is staying on the rod
- 7) When plugging the ID rod into the straw we advise to use a plugging guide (ref: 008200) that hold the straw straight while inserting the rod